



# How to Add a Cine Clip/Series of Multiple Images to PowerPoint

Using Volume Vision in Philips PACS


- Log into PACS
- Search for the exam and open it
- Right click on images, click on MPR, which will launch Volume Vision

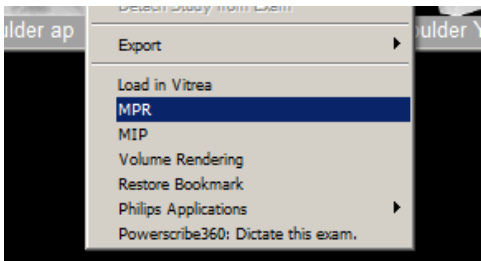


- Hover over the left side of the image you would like to save

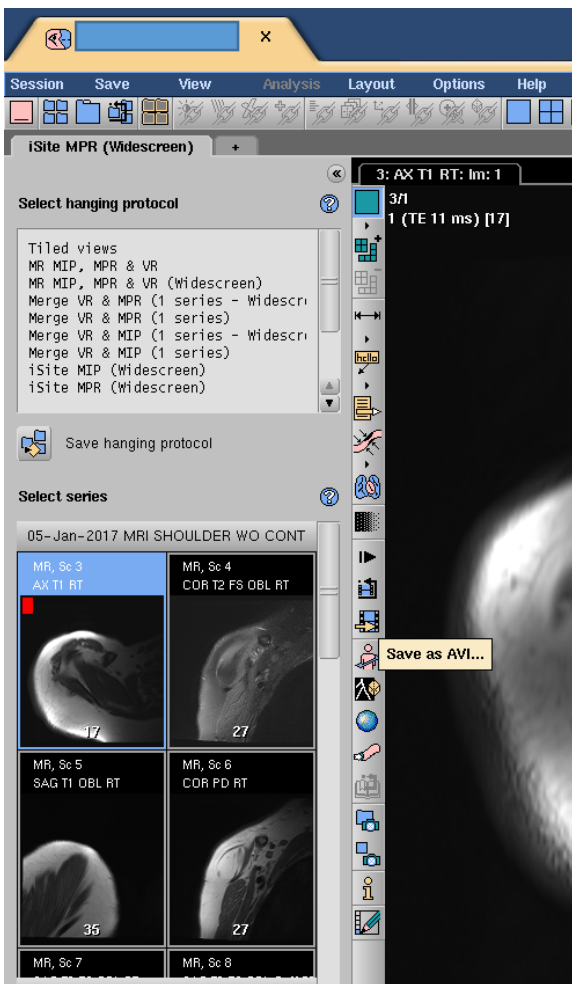


- Click on Save as AVI

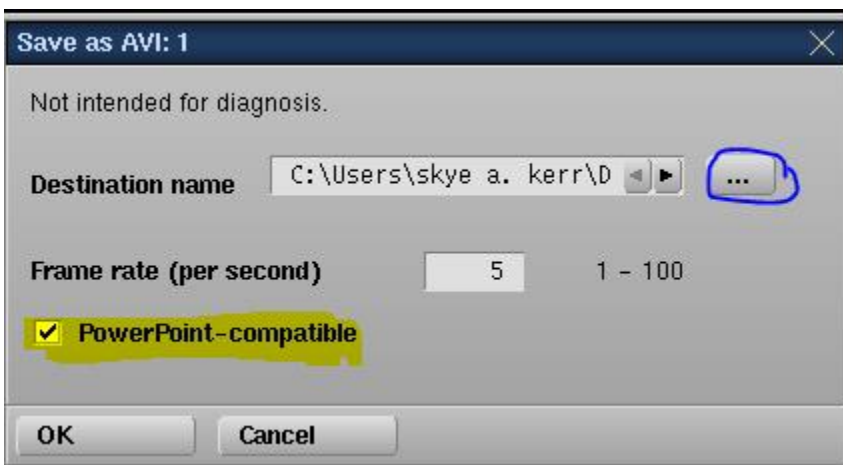


- Check the PowerPoint-compatible button
- Click the ellipse and browse to save file into your I-Drive, R-Drive folder, or the computer desktop

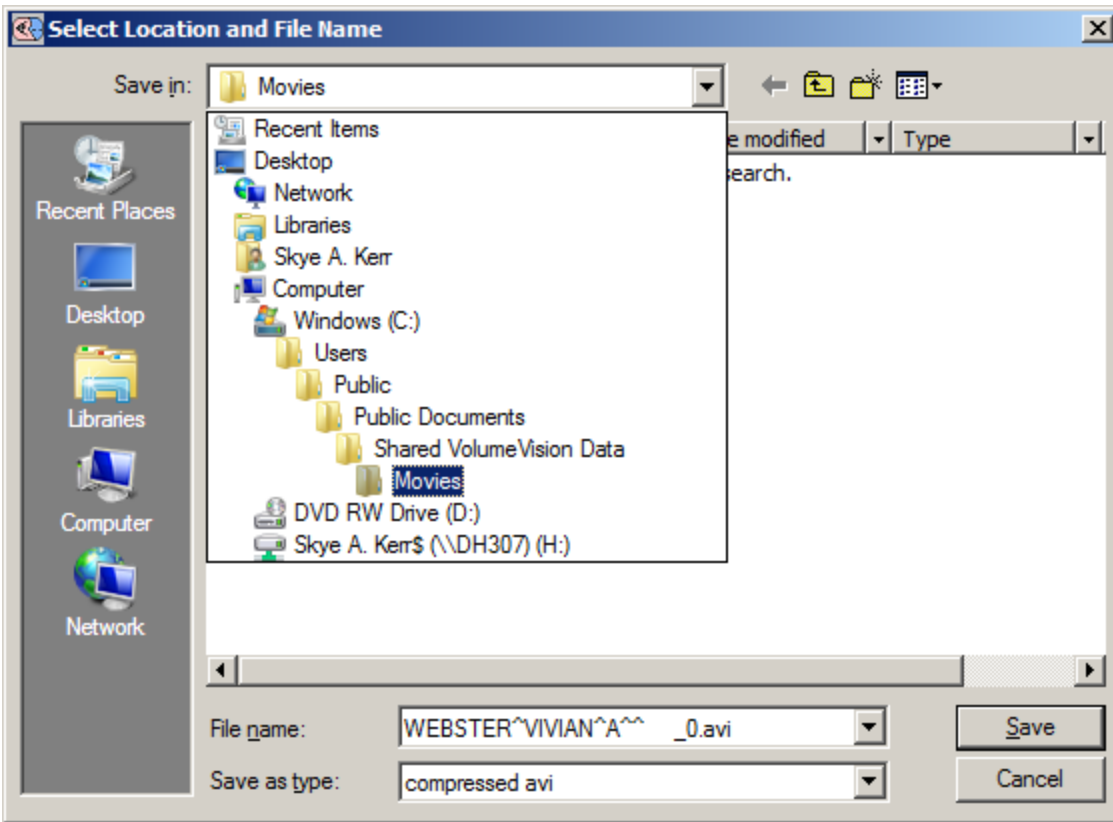

- Press ok
- Open PowerPoint. When you are ready drag and drop the file into the PowerPoint slide (or

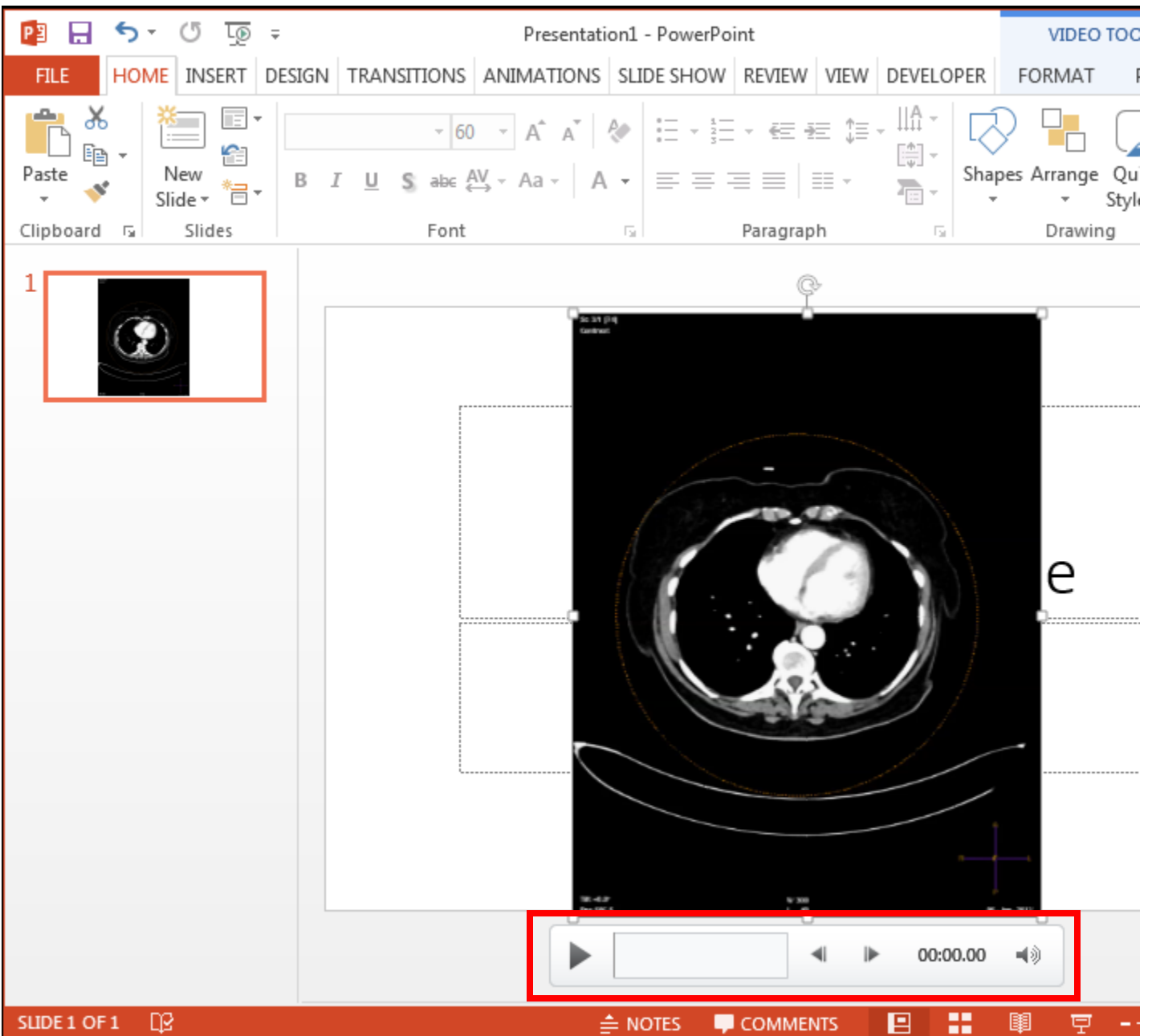

- When you are done in Volume Vision, click the X in the yellow bar

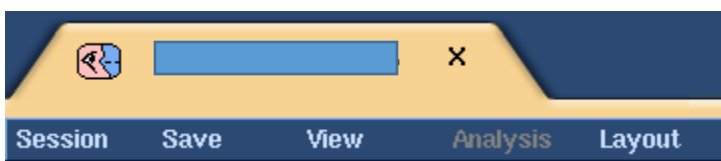

Unfortunately there is now way to anonymize images, but you can use Shapes to insert a block over the patient information.

